



PROFECT® 128 HDQ NEUTRAL

Highly Concentrated Neutral Disinfectant Cleaner



Profect® 128 HDQ Neutral is a highly concentrated, one step¹ hospital disinfectant cleaner¹ and deodorant formulated to kill a broad spectrum of microorganisms such as Human Rotavirus, Feline Picornavirus, and Vancomycin Resistant *Enterococcus faecalis* (VRE) on hard, nonporous environmental surfaces. At 1 fluid ounce per gallon dilution, Profect 128 HDQ Neutral is cost-effective and reduces restocking intervals for routine high touch surface cleaning. Use Profect 128 HDQ Neutral daily as part of a simple and effective cleaning and disinfection program⁸ for your entire facility.

EPA Registration Number: 5741-34

PROFECT® 128 HDO NEUTRAL

FEATURES

- Sanitizes⁹ in 120 seconds***
- Bactericide/Virucide*/Mildewstat
- Effective against antibiotic-resistant bacteria^{1,8,**}
- Meets surface disinfection recommendations of OSHA's Bloodborne Pathogens Standard

BENEFITS

- Disinfects, cleans, and deodorizes in one labor-saving step¹
- Economical to use - dilute at 1 fluid ounce per gallon of water
- Delivers powerful cleaning and excellent surface compatibility
- Leaves the room clean and disinfected^{1,8}

¹When used according to disinfection directions for all listed organisms

⁸On hard, non-porous surfaces

⁹When used as directed for sanitizing hard, non-porous, non-food contact surfaces

*Hepatitis C Virus (HCV), Hepatitis B Virus (HBV), HIV-1 (AIDS Virus), Influenza A Virus, and Influenza B Virus

***Staphylococcus aureus*-MRSA, *Enterococcus faecalis*-VRE and *Acinetobacter baumannii*

****Staphylococcus aureus* and *Enterobacter aerogenes*

SPECIFICATION DATA

Dilution	1:128
pH	6.0 – 9.0
Color	Red
Fragrance	Refreshing Citrus
Specific Gravity @ 24°C/75°F	1.00
Flash Point	69.2° – 72.2°C
Stability: Shelf @ 24°C/75°F	1-year minimum
Stability: Freeze/Thaw	3 cycles
EPA Registration Number	5741-34

ORDERING INFORMATION



#107204 #107205 #107255 #107260

EFFICACY DATA

FOR USE AS A ONE-STEP¹ DISINFECTANT, DEODORIZER, AND CLEANER ON HARD, NON-POROUS SURFACES

When used as directed, this product is effective against *Pseudomonas aeruginosa*, *Staphylococcus aureus*, *Bordetella bronchiseptica*, *Enterobacter aerogenes*, *Escherichia coli* O157:H7, Extended-Spectrum beta-lactamase (ESBL) producing *Escherichia coli*, *Klebsiella pneumoniae* (ESBL), *Klebsiella pneumoniae* (NDM-1), *Listeria monocytogenes*, Methicillin Resistant *Staphylococcus aureus* (MRSA), Multi-drug Resistant (MDR) *Acinetobacter baumannii*, *Neisseria gonorrhoeae*, *Proteus mirabilis*, *Proteus vulgaris*, *Salmonella enterica*, *Salmonella enterica* subspecies *enterica* serovar *Pullorum*, *Salmonella enterica* subspecies *enterica* serovar *Typhimurium*, *Serratia marcescens*, *Shigella flexneri* serotype 2B, *Shigella sonnei*, *Streptococcus pneumoniae*, *Streptococcus pyogenes*, Vancomycin Resistant *Enterococcus faecalis* (VRE), *Vibrio cholerae*, and *Yersinia enterocolitica*.

*VIRUCIDAL ACTIVITY ON HARD, NON-POROUS SURFACES

When used as directed, this product is effective against Hepatitis C Virus (HCV), Hepatitis B Virus (HBV), HIV-1 (AIDS Virus), Herpes Simplex Virus Type 1, Herpes Simplex Virus Type 2, Human Coronavirus, Human Rotavirus, Influenza A Virus, Influenza B Virus, SARS-CoV-2, and Vaccinia virus.

*ANIMAL PREMISE VIRUCIDAL ACTIVITY ON HARD, NON-POROUS SURFACES

When used as directed, this product is effective against Avian Influenza A Reassortant virus, Canine Distemper virus, Feline Picornavirus, Infectious Bovine Rhinotracheitis, Porcine Respiratory and Reproductive Syndrome (PRRS) virus, Pseudorabies virus, and Transmissible Gastroenteritis virus (TGE).

TO CONTROL MILDEW

When used as directed this product is an effective mildewstat against *Aspergillus niger*.

TO SANITIZE SOFT SURFACES

When used as directed this product is effective against *Enterobacter aerogenes* and *Staphylococcus aureus*.

NON-FOOD CONTACT SANITIZING ON HARD, NON-POROUS SURFACES

When used as directed for sanitization this product is effective against *Enterobacter aerogenes* and *Staphylococcus aureus* on hard, non-porous, non-food contact surfaces.

ADDITIONAL RESOURCES

Scan the QR code to access this product's safety information and documentation.



SAFETY: For institutional and industrial use only. Be sure to read all directions, precautionary and first aid statements on product labels before using this or any other Spartan product. If questions remain, consult your employer or a physician. Safety Data Sheets for all Spartan products are available from your authorized Spartan distributor and at www.spartanchemical.com. Workplace labels are available in English, Spanish, and French.

GUARANTEE: Spartan's modern manufacturing and laboratory control ensures uniform quality. If dissatisfied with performance of product, any unused portion may be returned for credit within 12 months of the date of manufacture.